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 Facebook
[scheu.dental](https://www.facebook.com/scheu.dental)

EN Instructions for use

Intended purpose: Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatments.

Indication: Suitable only for the fabrication of aligner splints, OSAMU-Retainers®, retainers, retention splits, long-term retainers and bite splints. Instructions for use can be found at www.scheu-dental.com/pressure-moulding-material.

Contraindication: Allergic reactions to components of IMPRELON® S/IMPRELON® S+ (copolyester) cannot be ruled out in sensitised people. Any use not stated above is contraindicated.

Storage: Store at 5°C to 35°C. Protect from sunlight. Keep dry.

Important: Must only be used by qualified personnel. Only open the barrier bag if it is undamaged and still sealed. For single use. Not suitable for build-up with resins. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately!

Safety information: The practitioner is responsible for correct use. The splint must not be used if the material is cracked/broken/ contains bubbles; there is a risk of swallowing or aspiration. When using IMPRELON® S with 3D-printed models, a non-perforated protective foil (ISOFOLAN®) or alternatively IMPRELON® S+ must be used. Remove the protective foil after the thermoforming process. Do not bring the products into contact with alcoholic/organic solvents.

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